


Minimum Tooth Reduction & Prep Guide

Prep recommendations for the most common restoration types.


IPS e.max® Press

O'Brien Enamelux®



Bonding is necessary to maintain the structural integrity of the restoration if the occlusal reduction is less than 1.5 mm.

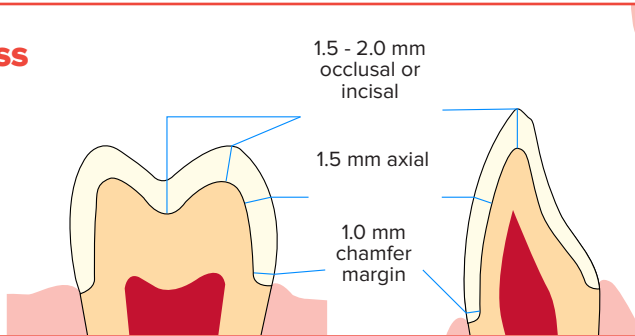

e.max® Veneer

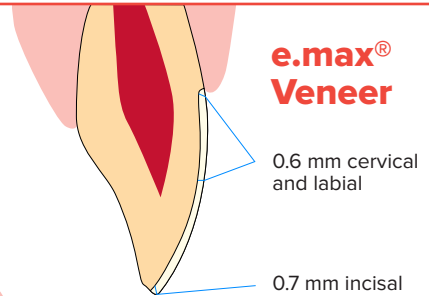

Porcelain Fused to Metal

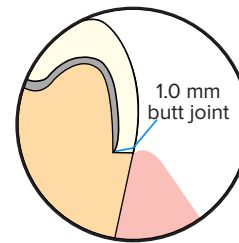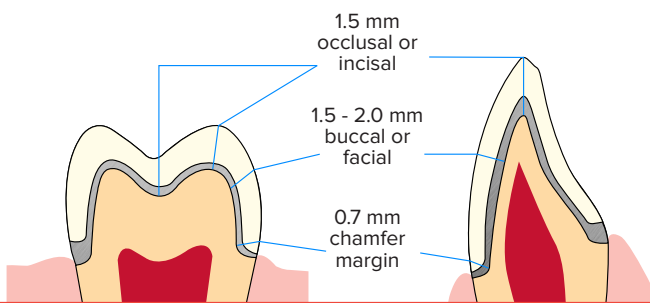


Porcelain Labial Margin


The addition of a PLM requires prepping the tooth with a 1.0 mm butt joint.

Monolithic Zirconia

O'Brien MZR™

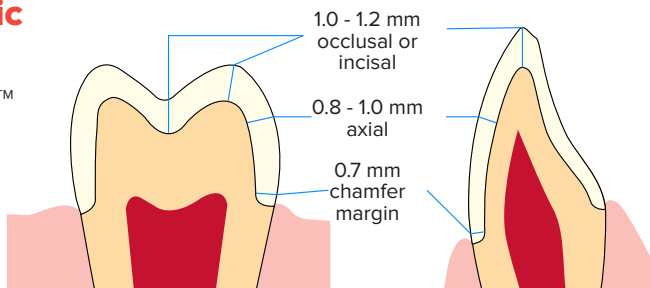


Full Cast Gold

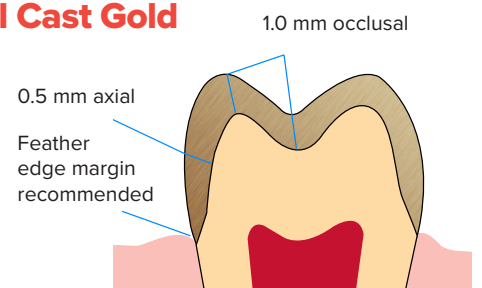

All Crowns Containing Ceramic

Make sure there are no sharp edges or corners as those create stress concentration points that can lead to cracks in the material.

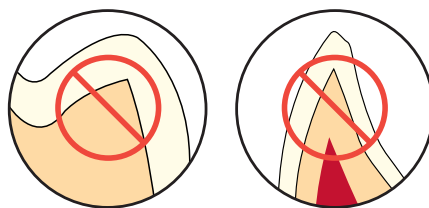

To Download This Guide

Go to obrientalab.com/prepguide or scan the QR code to the right to download a copy of this guide.



O'Brien
DENTAL LAB